

cPL Test Interpretation:

< 200 µg/L – Not consistent with pancreatitis

200-400 µg/L – Equivocal zone: pancreatitis possible, further evaluation may be warranted >

400 µg/L – Consistent with pancreatitis

Cobalamin/Folate Reference Ranges and Test Interpretation:

Serum Cobalamin and Folate Reference Ranges		
Analyte	Canine	Feline
Cobalamin (ng/L)	251 - 908	> 290
Folate (µg/L)	7.7 – 24.4	9.7 – 21.6

Clinical Interpretation of Serum Cobalamin & Folate Levels		
Serum Cobalamin	Serum Folate	Potential Diagnosis
Low	Normal	Ileum-localized disease
Normal	Low	Jejunum-localized disease
Low	Low	Diffuse small intestinal disease
Low	High	Exocrine Pancreatic Insufficiency (EPI) or Small Intestinal Bacterial Overgrowth (SIBO)

***NOTE:** If the denaturant step is not performed, results will indicate a markedly elevated folate with a low or low-normal cobalamin. If you see these results, please ensure that the denaturant step was performed appropriately.